Aquafix®

GB Intended use: Kessler Aquafix® sterile waterproof self adhesive wound dressings, are designed to effectively protect small and large wounds such as cuts, abrasions, surgical incisions and IV catheter sites as well as to absorb their secretions. Patient may bathe or shower without removing dressing.

Product characteristics: The dressing consists of a hyper absorbent wound pad that ensures reliable absorption of secretions, bonded on a larger breathable and flexible polyurethane film that ensures patient comfort as well as conformability to all wound sites. The cover is coated with hypoallergenic, acrylic adhesive. The wound pad has a special anti-adherent layer which prevents adhesion of the pad on the wound.

Instructions of use: Wash your hands before and after dealing with the wound. Clean and disinfect the wound in accordance with standard hospital protocol or according to your doctor recommendation. Skin should be clean and dry. Choose the appropriate dressing size and remove the individual package. Remove half paper liner from the adhesive side of the dressing and stick the exposed half strip over the wound, avoiding touching the self adhesive area. Then remove the second half of the paper liner and press on all sides in order to achieve complete waterproofing of the wound. Finally, remove both dorsal protective liners. Reapply as frequently as required. Renew every 48 hours or earlier if needed or as directed by your doctor. Remove it slowly and carefully by peeling one of the edges in the direction of hair growth.

Precautions: Read the instructions carefully before use. Single use only. Sterile unless package is opened or damaged. Seek medical advice immediately for all serious wounds and burns and if redness or discomfort occurs.

Warnings: Not to be used in case of known intolerance to similar products, inflammatory skin conditions as well as beyond product validity stated on the packaging.

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