GB Intended use: Kessler Ocupad® sterile self adhesive ocular dressings are designed to effectively protect the eye area after surgery or injury. The dressing's anatomic form in combination with its soft and flexible material ensures maximal comfort for the patient.

Product characteristics: The dressing consists of a hyper absorbent wound pad that ensures reliable absorption of secretions, bonded on a larger soft, breathable and flexible non woven cover that ensures patient comfort. The cover is coated with hypoallergenic, acrylic adhesive. The wound pad has a special anti-adherent layer which prevents adhesion of the pad on the wound.

Instructions of use: Wash your hands before and after dealing with the wound. Ocupad® should be applied to clean and dry skin. Remove the individual packaging. Apply Ocupad® starting just below the eyebrow. The narrow tip should be placed towards the nose. Reapply every 48 hours or earlier if needed or as directed by your doctor. Change the dressing if secretions penetrate the soft cover. Remove slowly and carefully from the edges towards the center.

Precautions: Read the instructions carefully before use. Single use only. Sterile unless package is opened or damaged. Seek medical advice immediately if redness or discomfort occurs.

Warnings: Not to be used in case of known intolerance to similar products, inflammatory skin conditions as well as beyond product validity stated on the packaging.

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