GB Intended use: Kessler Primafix® sterile adhesive wound dressings, are designed to effectively protect small and large wounds such as cuts, burns, abrasions, surgical incisions and IV catheter sites as well as to absorb their secretions.

Product characteristics: The dressing consists of a hyper absorbent wound pad that ensures reliable absorption of secretions, bonded on a larger soft, breathable and flexible non woven cover that ensures patient comfort as well as conformability to all wound sites. The cover is coated with hypoallergenic, acrylic adhesive. The wound pad has a special anti-adherent layer which prevents adhesion of the pad on the wound.

Instructions of use: Wash your hands before and after dealing with the wound. Clean and disinfect the wound in accordance with standard hospital protocol or according to your doctor recommendation. Choose the appropriate dressing size and remove the individual package. Peel the paper liner and apply it to clean and dry skin. Reapply every 48 hours or earlier if needed or as directed by your doctor. Change the dressing if secretions penetrate the soft cover. Remove it slowly and carefully by peeling one of the edges in the direction of hair growth.

Precautions: Read the instructions carefully before use. Single use only. Sterile unless package is opened or damaged. Seek medical advice immediately for all serious wounds and burns and if redness or discomfort occurs.

Warnings: Not to be used in case of known intolerance to similar products, inflammatory skin conditions as well as beyond product validity stated on the packaging.

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