GB Intended use: Kessler® Steropad sterile, super-absorbent, non-stick dressings are designed to effectively protect small and large wounds such as cuts, burns, abrasions, surgical incisions, etc. as well as to absorb moderately to heavy secretions without sticking to the wound.

Products characteristics: The dressing consists of an absorbent core made by polyester/rayon material, which is coated on both sides by a smooth, perforated, non-stick polyester film, which prevents the core to stick to the wound. The dressing can be cut in order to fit various wound sites. Latex free.

Instructions of use: Wash your hands before and after dealing with the wound. Clean and disinfect the wound in accordance with standard hospital protocol or according to your doctor recommendation. Choose the appropriate dressing size, peel open the packaging and place it on the wound (can be secured in place with medical tape, such as Enoplast, Enosilk, Enopore, Enofilm). Renew every 48 hours or earlier if needed or as directed by your doctor. Change the dressing if secretions penetrate it.

Precautions: Read the instructions carefully before use. Single use only. Sterile unless package is opened or damaged. Seek medical advice immediately for all serious wounds and if redness or discomfort occurs.

Warnings: Not to be used in case of known intolerance to similar products as well as beyond product validity stated on the packaging.

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